



Grip-Lok[®]

Suture-free Fixation System for Arrow CVC/PICC Catheters

Fixation made safe and easy

The Grip-Lok[®] Fixation System is one element of the global CVC/PICC concept developed for Teleflex for managing the risk of infection associated with catheterisation.

Grip-Lok[®] is a suture-free fixation system that holds the catheter in place securely, whatever the access. The soft, pre-cut, hub-shaped component makes it easier to position

the catheter and ensures fixation and stabilisation.


- Soft, flexible design for patient comfort
- Provides catheter stabilisation during accidental infusion-line pulls
- Easy to apply, inspect and adjust



Suture-free fixation Grip-Lok[®]



The soft, pre-cut, hub-shaped component ensures fixation and stabilisation

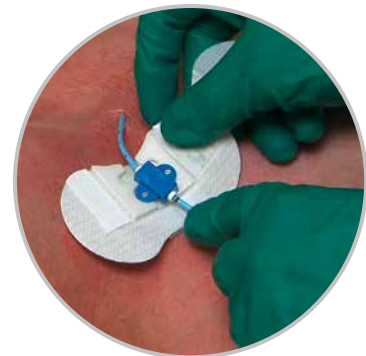

The soft, pre-cut, hub-shaped component also fits the catheter's additional fixation wings

Grip-Lok®: Fast Facts

Safe

Easy application

Hypoallergenic

Breathable and made without natural latex rubber to reduce the risk of allergic reactions and skin irritation

Positioning the Grip-Lok® Suture-free Fixation System for Arrow CVCs and PICCs



Prepare the skin in accordance with established practice.

Preparation or hair removal may be required for better adhesion.

- ① Open the top flap and slide the Grip-Lok® under the catheter hub. Position the hub in the corresponding adhesive area and press gently onto the adhesive.
- ② Lift and remove the liner labeled "Pull" on the flap to expose the adhesive.
- ③ Re-close the flap on the catheter hub and press onto the adhesive. The self-gripping part of the flap can be repositioned, allowing the catheter to be inspected and adjusted.
- ④ Lift and remove the bottom liner from one side while holding the Grip-Lok® in place. Do the same on the other side of the device.

After securing the Grip-Lok®, finish dressing in accordance with best current practice. Change the transparent adhesive dressing using the stretch technique.

Fixation System

REF.	DESCRIPTION	QTY
3604MCS-TA-1	Suture-free fixation system for Arrow CVCs and PICCs	50

Grip-Lok is a registered trademark of TIDI Products, Inc. Teleflex, the Teleflex logo and Arrow are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative.

Revised: 08/2017. © 2017 Teleflex Incorporated. All rights reserved. 94 10 93 - 00 00 01 - REV B - 08 17 01

Teleflex

Distributed by:
Teleflex Headquarters International, Ireland · Teleflex Medical Europe Ltd. · IDA Business & Technology Park
 Dublin Road · Athlone · Co Westmeath · Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.intl@teleflex.com
United Kingdom · Tel. +44 (0)14 94 53 27 61 · info.uk@teleflex.com
South Africa · Tel. +27 (0)11 807 4887 · assist.africa@teleflex.com