



# RÜSCH EASYCRIC

## Seldinger Based Emergency Cricothyrotomy Set

### AT A GLANCE

- complete Seldinger-based cricothyrotomy set for adults (size 5)
- premounted EasyCric tube and dilator (hydrophilic coated, anatomically shaped)
- ergonomic grip
- well aligned set components, e.g. puncture needle, syringe, saline pouch
- latex-free
- sterile
- single use

### FOCUS ON SAFETY AND PERFORMANCE – WHEN EVERY SECOND COUNTS


Cricothyrotomy is a last resort potentially life-saving emergency procedure when immediate access to the patient’s otherwise non-accessible airway is to be obtained. Quick-time decision, speediness, safe and accurate performance are critical in one of the most stressful situations for the clinician.

Introducing Teleflex’s RÜSCH brand EasyCric emergency cricothyrotomy set invented by Prof. Giulio Frova, this procedure can be made safer, easier and more efficient and thus help to improve patient outcome<sup>1</sup>.

The unique EasyCric features a Seldinger-based guide-wire technique for safe and accurate placement.

Thanks to the hydrophilic coating and anatomical design of the entire device (premounted tube and dilator, ergonomic grip, fixed neck-plate), an easy and atraumatic insertion as well as safe handling for the clinician can be secured. Well-aligned accessories such as a puncture needle, a syringe and the saline solution to perform the air aspiration test via the syringe & to activate the hydrophilic coating, complete the set.

EasyCric emergency cricothyrotomy set is a back-up device when every other procedure is impossible. Having a set readily available at all times may save a patient’s life.

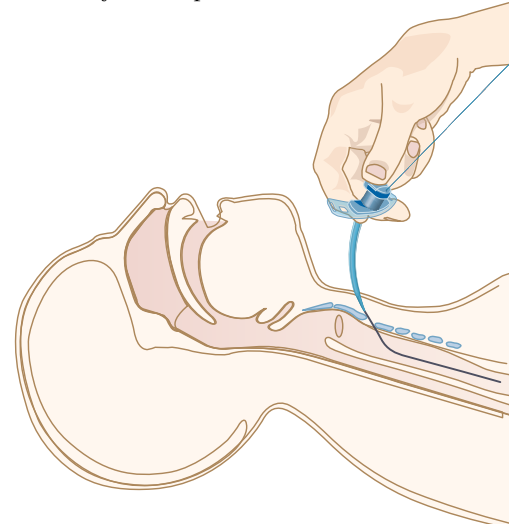

<sup>1</sup>(G.Frova, G. Merli, M. Sorbello, “First evaluation of a new cricothyrotomy set. Comparison of Melker, Minitrach II S, Quicktrach and EasyCric in a mannequin”. ESA 2010, 19APS – 10, 267.)



#### SET COMPONENTS

- ① EasyCric Tracheostomy Tube, transparent, 15 mm connector & fixed transparent neckplate, pre-mounted on the EasyCric Insertion Dilator, blue
- ② Guide-wire (non-kinking, straight soft tips, 40 cm long, Ø1.3 mm)
- ③ Comfort neck band
- ④ Puncture needle (3 cm)
- ⑤ Scalpel
- ⑥ Luer-lock syringe (5 ml)
- ⑦ 20 ml saline solution

#### EASYCRIC EMERGENCY CRICOTHYROTOMY TUBE SET (Ref. 120605)

ORDER SIZE	ID BEHIND NECKPLATE IN MM	ID AT TIP IN MM	OD BEHIND NECK PLATE IN MM	OD AT TIP IN MM	L (CENTRE LINE) IN MM	A IN MM	B IN MM	C IN MM	TOTAL LENGTH IN MM	ANGLE IN DEGREE°	QTY
5.0	6.0	5.0	8.5	5.8	70.0	0	0	70.0	95.0	66.9°	1








#### INSERTION DILATOR

ID AT GRIP IN MM	ID AT TIP IN MM	OD AT TIP IN MM	LENGTH BEHIND GRIP TO TIP IN MM	TOTAL LENGTH IN MM	ANGLE IN DEGREE°
3.0	1.5	2.0	125.0	130.0	66.9°



Blue insertion dilator with ergonomically shaped grip area for safer handling.

#### DIMENSIONS ACC. TO DIN EN ISO 5366-1

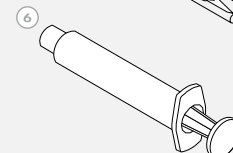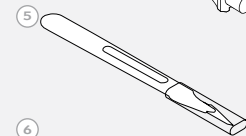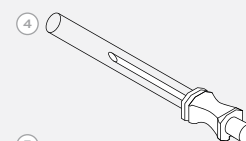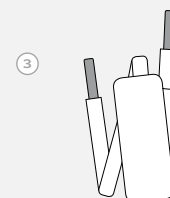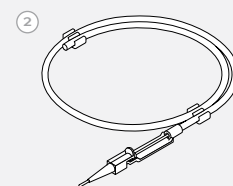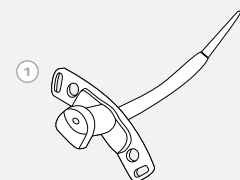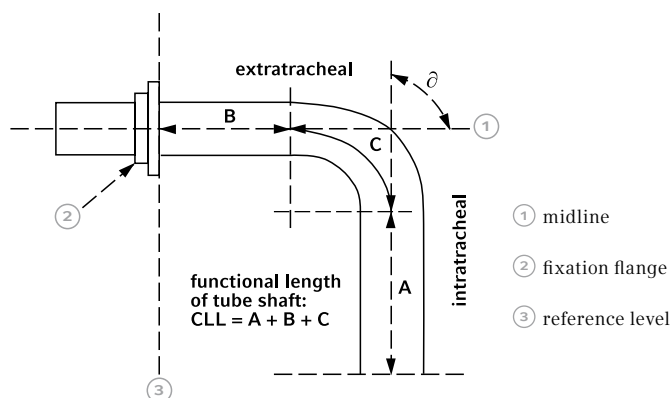

#### DISTRIBUTED BY:

TELEFLEX MEDICAL HEADQUARTER EMEA, IRELAND · IDA Business & Technology Park · Athlone · Co Westmeath

Tel. +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73 · orders.intl@teleflex.com

UNITED KINGDOM Tel. +44 (0)14 94 53 27 61 · info.uk@teleflex.com

SOUTH AFRICA Tel. +27 (0)11 807 4887 · assist.africa@teleflex.com

WWW.TELEFLEX.COM

94 06 37 - 00 00 01 · REV A · 07 12 02 · All data current at time of printing. Subject to technical changes without further notice.