



Weck

EFx Shield Fascial Closure System

Safety and Speed in Closure



Weck EFX Shield Fascial Closure System

The EFX Shield System offers speed and safety with an array of enhanced features

The Weck EFX Shield Fascial Closure System from Teleflex is the only shielded port closure device, providing enhanced sharps protection for safe, uniform, and consistent fascial closure.

Features

- Unique shielded wing design for enhanced sharps protection
- Intuitive wing deployment
- Innovative suture retrieval system for unassisted fascial closures

Suture retriever button
designed to make capture and retrieval of suture strands fast and secure


Reset button

Finger grip pads
ergonomic device control

Suture retriever exit line
proper depth of suture positioning

Retraction wings
for substantial retraction and improved stability on the abdominal wall

Suture retrieval wing shields
provide enhanced sharps protection, safe suture retrieval, and accurate placement 1 cm lateral to the defect midline

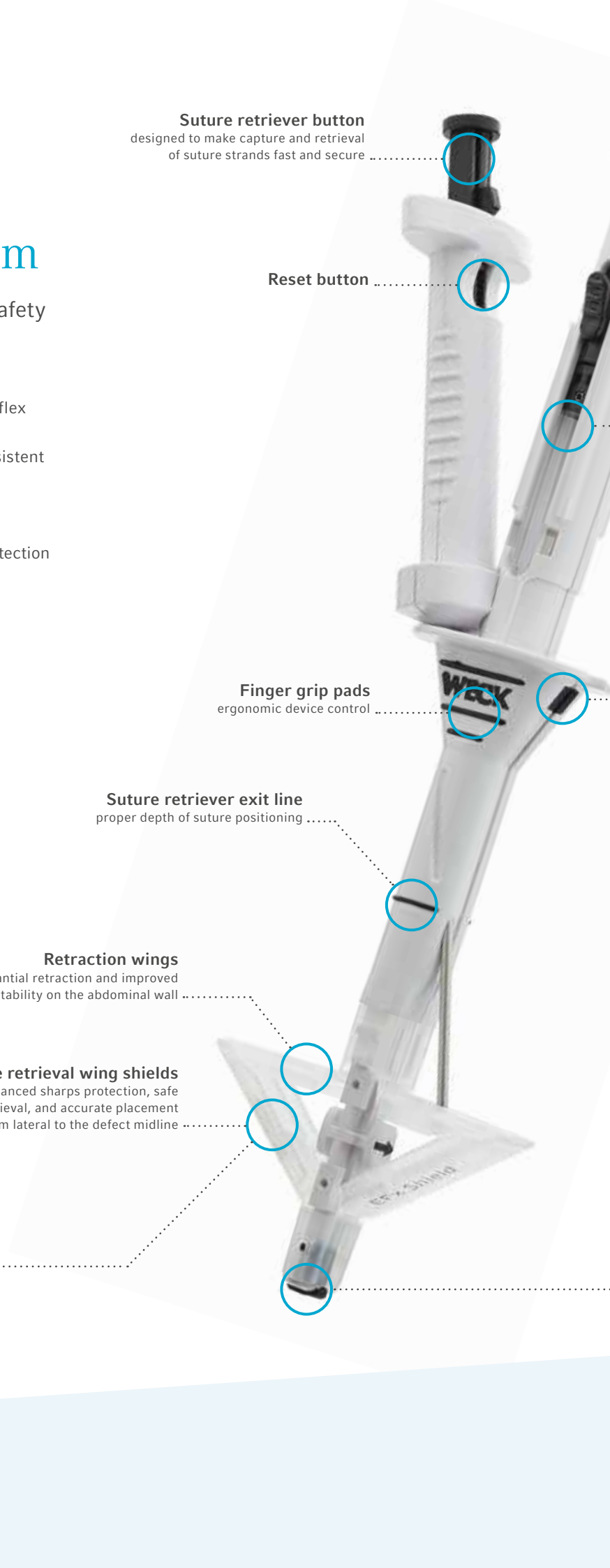




Slide lock
makes wing deployment
and approximation easy


Fast, safe suture retrieval

To prepare the system, remove both components from the packaging and load the EFx Shield device with a size 0, 18-inch absorbable suture.

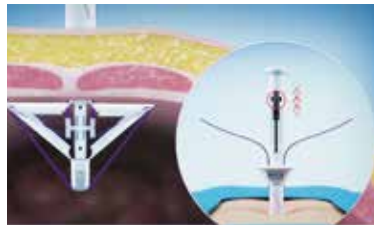


Step 1: Deploy the device

Insert the device into the defect and use the slide lock to engage wing shields. Once the wings are locked, position them against the abdominal wall. The suture retriever exit line should be below skin level to ensure an optimal suture purchase.

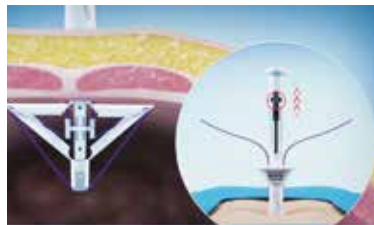

Step 2: Retrieve the suture

Fully insert the suture retriever into the guide channel and press the black suture retrieval button until it locks to capture the suture. Remove the retriever and press the reset button to repeat the process on the other side.

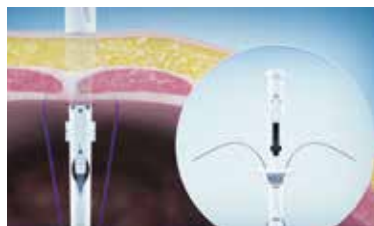

Step 3: Collapse and close

Reposition the slide lock to collapse the wing shields, then remove the device from the defect. Secure both ends of the suture and tie using your preferred method to complete the closure.

Suture anchors
enable quick and secure
suture loading for ease of use

Ordering Information

REF.	DESCRIPTION	QTY.
EFX002	Weck EFx Shield Fascial Closure System	5/box

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit teleflex.com.

Teleflex is the home of Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüsçh and Weck – trusted brands united by a common sense of purpose.

Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

Regional Offices

United States: Phone +1 919 544 8000, Toll Free 866 246 6990, cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

Latin America: Phone +1 919 433 4999, la.cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

International: Phone +353 (0)9 06 46 08 00, orders.intl@teleflex.com, Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Australia/New Zealand 1300 360 226

Austria +43 (0)1 402 47 72

Belgium +32 (0)2 333 24 60

Canada +1 (0) 905 943 9000

China (Shanghai) +86 (0)21 6163 0965

China (Beijing) +86 (0)10 6418 5699

Czech Republic +420 (0)495 759 111

France +33 (0)5 62 18 79 40

Germany +49 (0)7151 406 0

Greece +30 210 67 77 717

India +91 (0)44 2836 5040

Italy +39 0362 58 911

Japan +81 (0)3 6632 3600

Korea +82 2 536 7550

Mexico +52 55 5002 3500

Netherlands +31 (0)88 00 215 00

Portugal +351 22 541 90 85

Singapore (SEA non-direct sales countries) +65 6439 3000

Slovak Republic +421 (0)3377 254 28

South Africa +27 (0)11 807 4887

Spain +34 918 300 451

Switzerland +41 (0)31 818 40 90

United Kingdom +44 (0)1494 53 27 61

For more information, please visit teleflex.com.

Teleflex, the Teleflex logo, EFX Shield and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this document is not a substitute for the product Instructions for Use. The products in this catalogue may not be available in all countries. Please contact your local representative. All data current at time of printing (10/2015). Subject to technical changes without further notice.
© 2015 Teleflex Incorporated. All rights reserved.