


Chocolate XD® PTCA Balloon Catheter

Predictable by design

Chocolate XD® PTCA Balloon Catheter

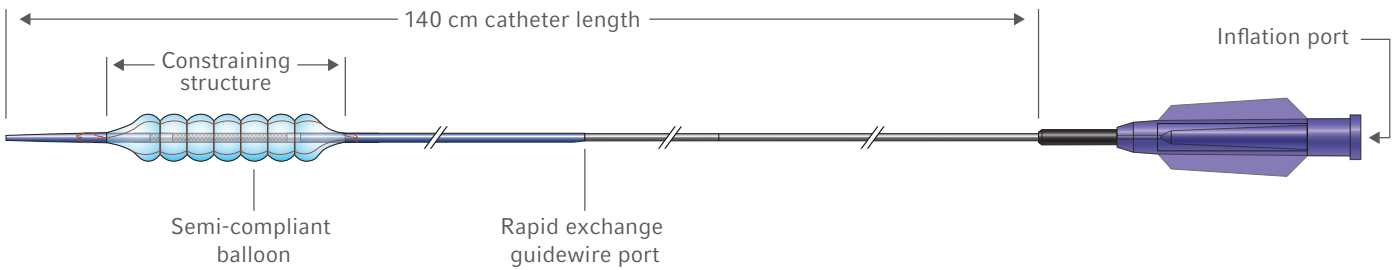

Ordering Information

Chocolate XD® PTCA Balloon Catheter

The Chocolate XD® PTCA Balloon Dilatation Catheter is indicated for balloon dilatation of the stenotic portion of coronary artery or bypass graft stenosis for the purpose of improving myocardial perfusion including in-stent restenosis. The Chocolate XD® PTCA Balloon Dilatation Catheter is contraindicated for unprotected left main coronary artery, coronary artery spasm in the absence of significant stenosis and crossing through stent struts.


CATALOG NUMBER	GUIDEWIRE COMPATIBILITY	CATHETER LENGTH	BALLOON DIAMETER	CROSSING PROFILE*	BALLOON LENGTH	NOMINAL PRESSURE	RATED BURST PRESSURE	GUIDE CATHETER COMPATIBILITY
CX14-140-2010 RX	0.014"	140 cm	2.0 mm	0.039" (1.00 mm)	10 mm	10 atm	16 atm	5 Fr.
CX14-140-2015 RX	0.014"	140 cm	2.0 mm	0.039" (1.00 mm)	15 mm	10 atm	16 atm	5 Fr.
CX14-140-2020 RX	0.014"	140 cm	2.0 mm	0.039" (1.00 mm)	20 mm	10 atm	16 atm	5 Fr.
CX14-140-2510 RX	0.014"	140 cm	2.5 mm	0.040" (1.02 mm)	10 mm	10 atm	16 atm	5 Fr.
CX14-140-2515 RX	0.014"	140 cm	2.5 mm	0.040" (1.02 mm)	15 mm	10 atm	16 atm	5 Fr.
CX14-140-2520 RX	0.014"	140 cm	2.5 mm	0.040" (1.02 mm)	20 mm	10 atm	16 atm	5 Fr.
CX14-140-3010 RX	0.014"	140 cm	3.0 mm	0.042" (1.07 mm)	10 mm	10 atm	16 atm	6 Fr.
CX14-140-3015 RX	0.014"	140 cm	3.0 mm	0.042" (1.07 mm)	15 mm	10 atm	16 atm	6 Fr.
CX14-140-3020 RX	0.014"	140 cm	3.0 mm	0.042" (1.07 mm)	20 mm	10 atm	16 atm	6 Fr.
CX14-140-3510 RX	0.014"	140 cm	3.5 mm	0.043" (1.08 mm)	10 mm	10 atm	16 atm	6 Fr.
CX14-140-3515 RX	0.014"	140 cm	3.5 mm	0.043" (1.08 mm)	15 mm	10 atm	16 atm	6 Fr.
CX14-140-3520 RX	0.014"	140 cm	3.5 mm	0.043" (1.08 mm)	20 mm	10 atm	16 atm	6 Fr.

Packaged in quantities of one unit per box.

*Crossing profile is defined as the maximum diameter found between the proximal end of the balloon and the distal tip of the catheter. The numbers above represent the average crossing profile (N=15 units) of each Chocolate XD® PTCA Balloon Catheter diameter.


Expand Your Expectations

The Chocolate XD® PTCA Balloon Catheter features a unique nitinol-constraining structure that is designed to uniformly expand to minimize stress on the vessel wall and reduce risk for dissection. Expand your expectations with a catheter that's designed to deliver predictable, atraumatic dilatation.

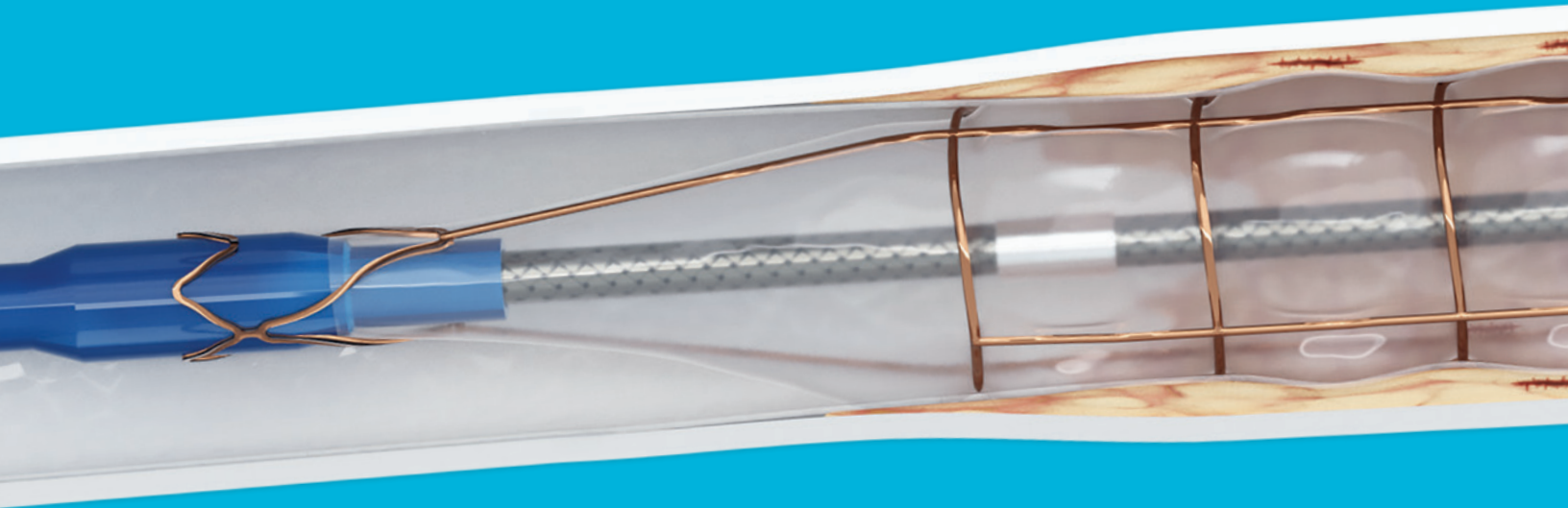



Designed for controlled dilatation

Nitinol-constraining structure is designed to facilitate uniform, atraumatic balloon expansion.

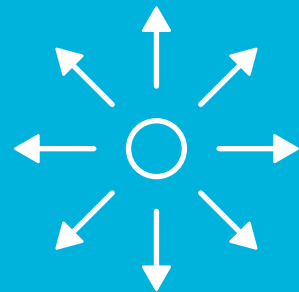

Designed to minimize vessel wall trauma

“Grooves” are designed to allow for stress relief and “pillows” are designed to provide multi-focal dilatation forces intended to reduce traumatic angioplasty effects.

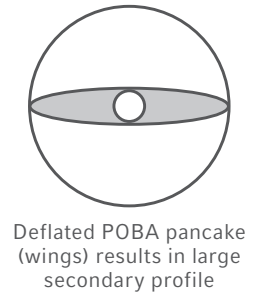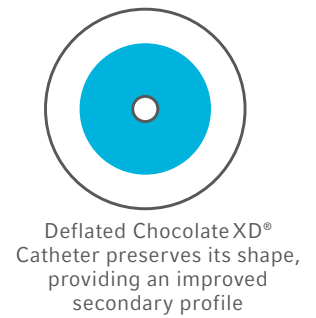

“Pillows”

“Grooves”




Designed to minimize “dog-boning”

The nitinol-constraining structure is designed to encourage synchronous full length inflation while preventing over expansion at the lesion shoulders.



Chocolate XD® PTCA Balloon


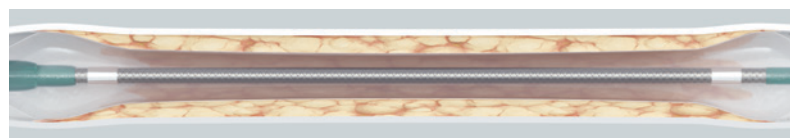


Standard PTCA Balloon

Designed for optimal vessel prep

The Chocolate XD® PTCA Balloon Catheter in combination with IVUS and Orbital Atherectomy (OA), facilitates stent delivery and optimal stent expansion, resulting in a large minimum stent area (MSA) in severely calcified coronary lesions.¹

1. Desai R, Mirza O, Martinsen BJ, et al. Plaque modification of severely calcified coronary lesions via orbital atherectomy: single center observations from a complex Veterans Affairs cohort. *Health Sci Rep.* 2018;1:e99.

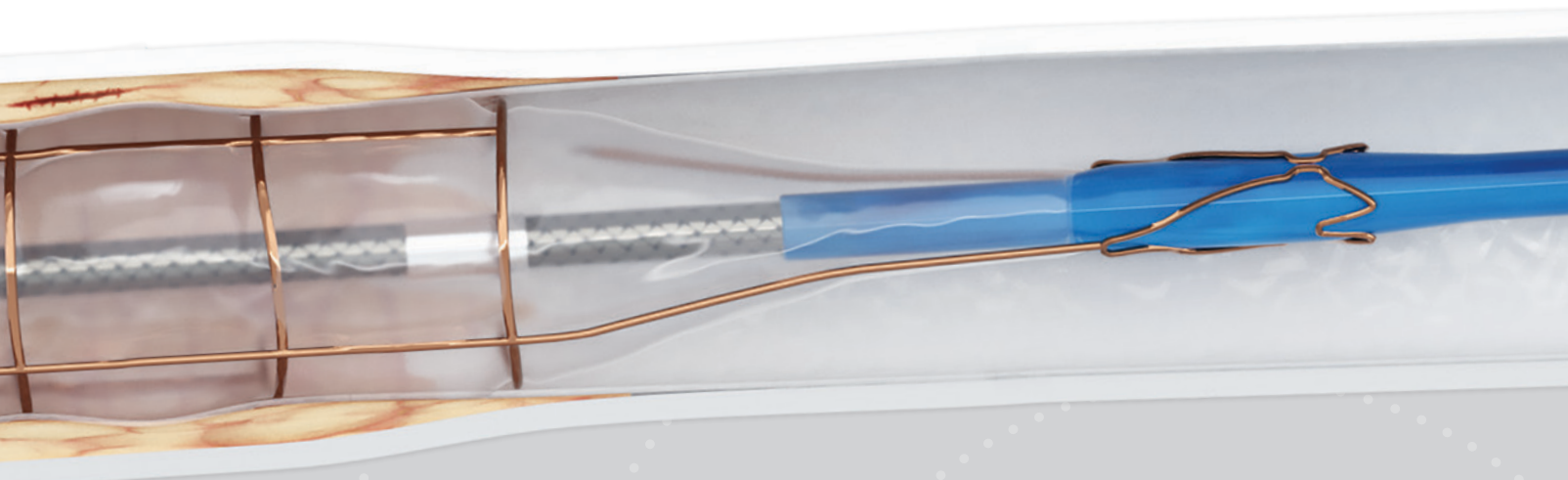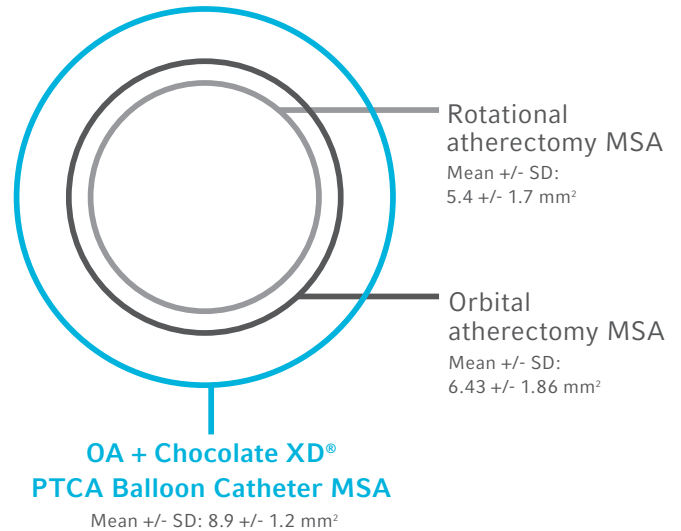

Expand Your Treatment Options

The Chocolate XD® PTCA Balloon Catheter is designed to offer a less traumatic alternative to conventional balloon angioplasty. It is designed to inflate uniformly to minimize stress on the vessel wall and can expand treatment options in no-stent situations:



Dual antiplatelet therapy adverse patients




Side branches



Small or distal vessels



Ostial lesions

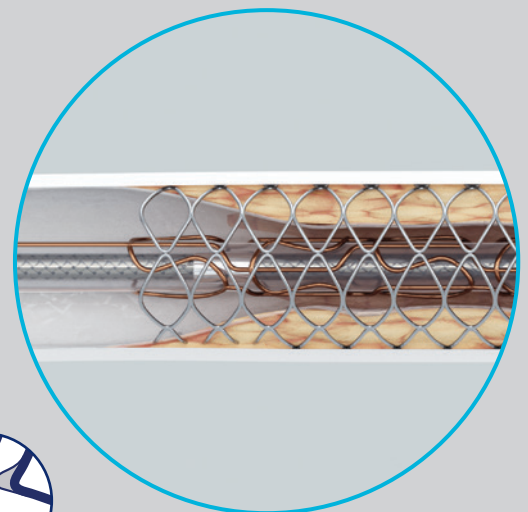

In-stent restenosis

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit teleflex.com.

Teleflex is the home of Arrow®, Deknatel®, Hudson RCI®, LMA®, Pilling®, Rüschi®, and Weck® – trusted brands united by a common sense of purpose.

Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

Regional Offices

United States: Phone +1 919 544 8000, Toll Free 866 246 6990, cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

Latin America: Phone +1 919 433 4999, la.cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

International: Phone +353 (0)9 06 46 08 00, orders.intl@teleflex.com, Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Australia 1300 360 226

Austria +43 (0)1 402 47 72

Belgium +32 (0)2 333 24 60

Canada +1 (0)800 387 9699

China (Shanghai) +86 (0)21 6163 0965

China (Beijing) +86 (0)10 6418 5699

Czech Republic +420 (0)495 759 111

France +33 (0)5 62 18 79 40

Germany +49 (0)7151 406 0

Greece +30 210 67 77 717

India +91 (0)44 2836 5040

Italy +39 0362 58911

Japan +81 (0)3 6632 3600

Korea +82 2 536 7550

Mexico +52 55 5002 3500

Netherlands +31 (0)88 00 215 00

New Zealand 0800 601 100

Poland +48 22 4624032

Portugal +351 22 541 90 85

Singapore (SEA non-direct sales countries) +65 6439 3000

Slovak Republic +421 (0)3377 254 28

South Africa +27 (0)11 807 4887

Spain +34 918 300 451

Switzerland +41 (0)31 818 40 90

United Kingdom +44 (0)1494 53 27 61

For more information, please visit teleflex.com.

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Chocolate XD is a registered trademark of TriReme Medical LLC.

Teleflex, the Teleflex logo, Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüschi and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. All other trademarks and registered trademarks are property of their respective owners.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 07/2019.

© 2019 Teleflex Incorporated. All rights reserved. MC-005667 LA EN