

TrapLiner[®] Catheter


Guide Extension Plus Wire Trapping

Feature Comparison Overview

TrapLiner® Catheter

6 Fr. 7 Fr. 8 Fr.


Advance guide extension up to 10 cm

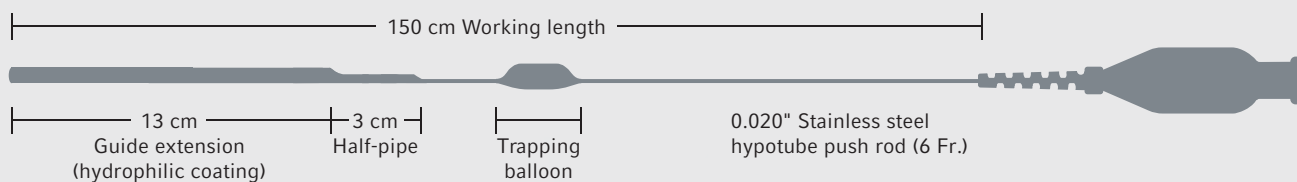

GuideLiner® V3 Catheter

5 Fr. 5.5 Fr. 6 Fr. 7 Fr. 8 Fr.

Advance guide extension up to 10 cm

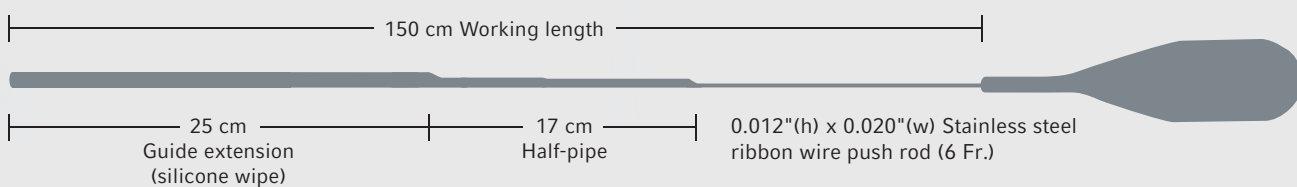

2-in-1 Device


Guide Extension Catheter With Trapping Capabilities


The TrapLiner® Catheter is a rapid exchange guide extension catheter that combines the ability to provide backup support with the ability to trap a 0.014" guidewire against the inner wall of a guide catheter. This functionality helps prevent the loss of guidewire position or unintentional guidewire advancement during catheter exchange, and eliminates the need for alternative catheter exchange techniques.

Guide Extension Clinical Benefits

Provides coaxial alignment and deep-seating for:

- Added backup support
- Distal device delivery
- Selective contrast delivery

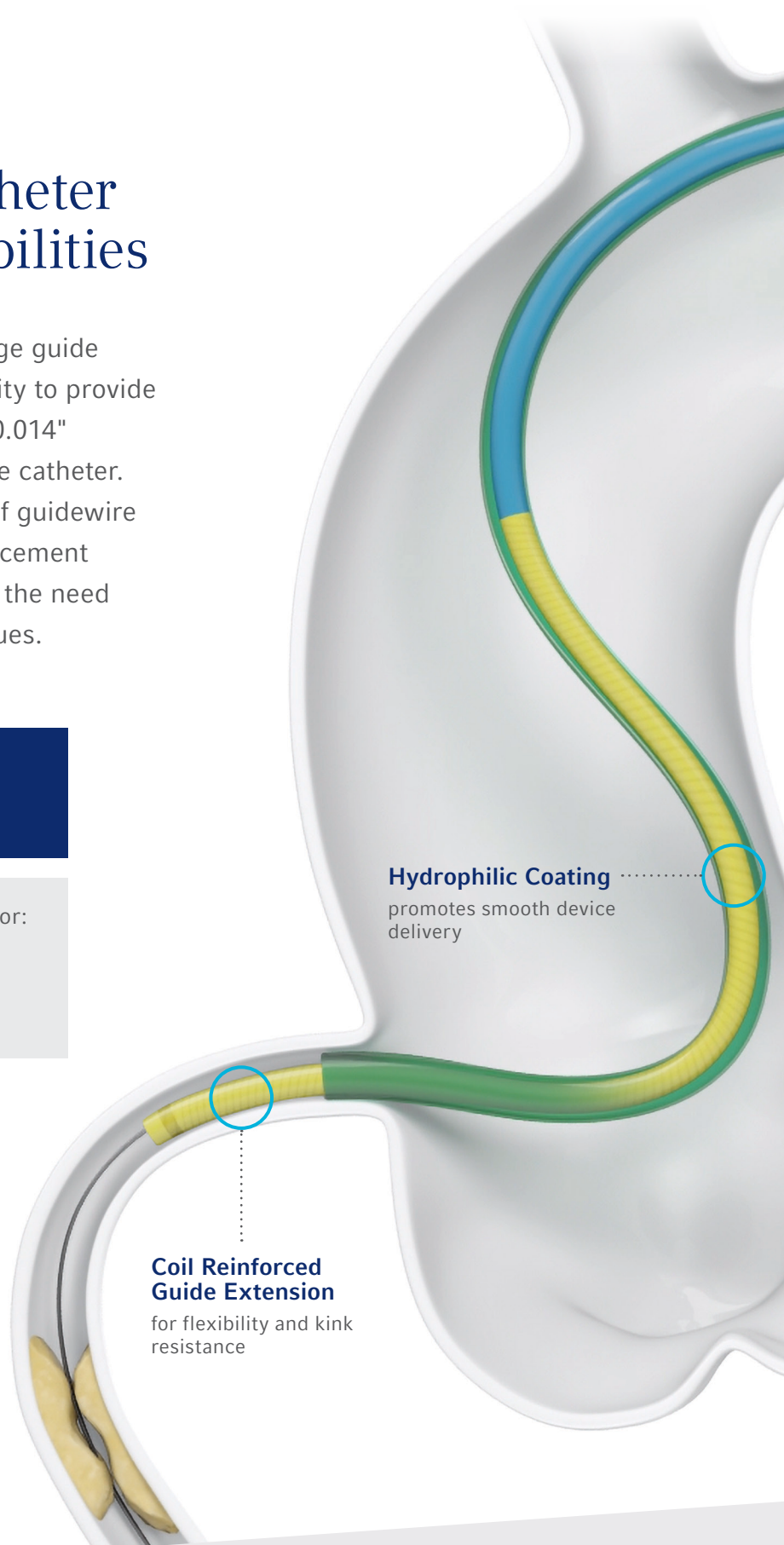

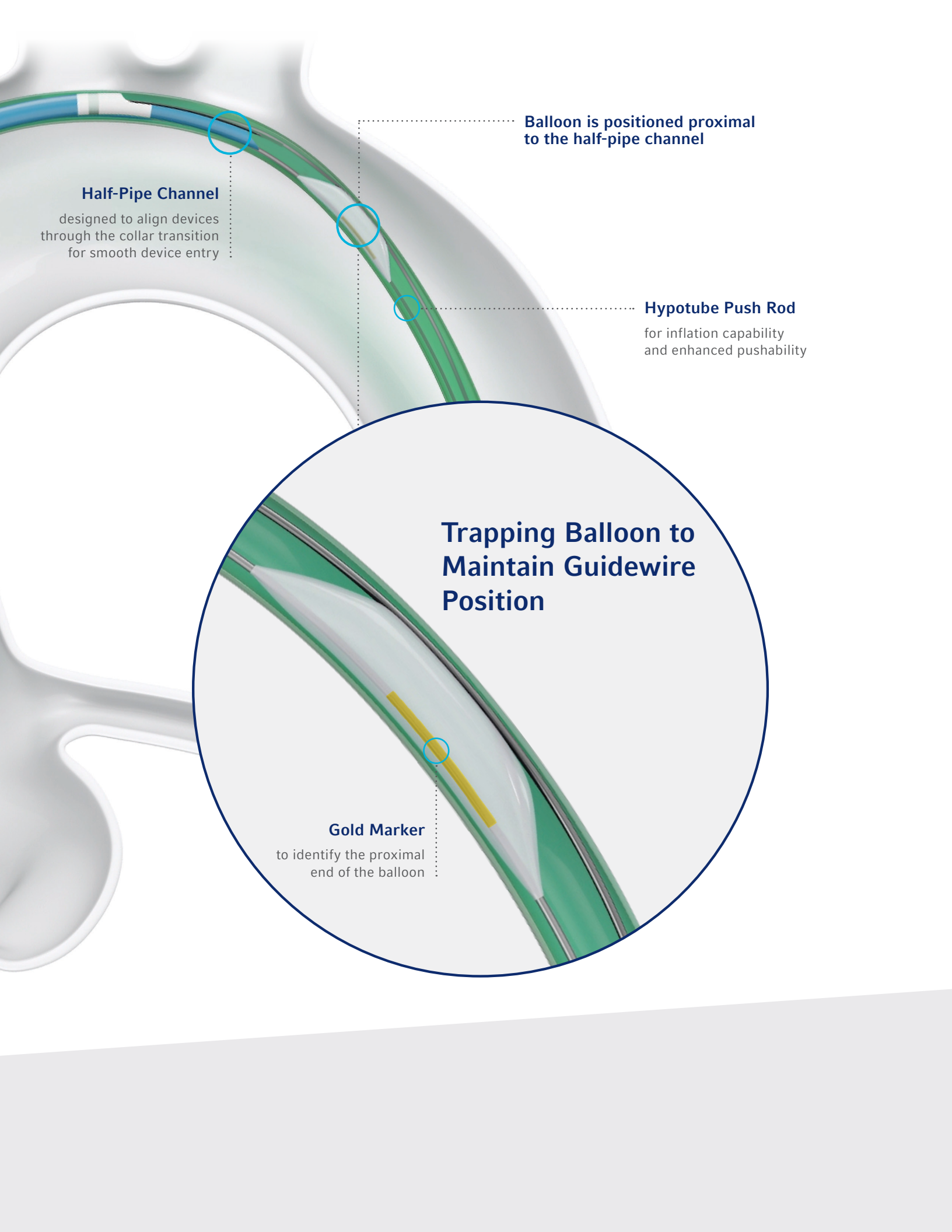

Half-Pipe Channel

designed to align devices through the collar transition for smooth device entry

Balloon is positioned proximal to the half-pipe channel

Hypotube Push Rod

for inflation capability and enhanced pushability

Trapping Balloon to Maintain Guidewire Position

Gold Marker

to identify the proximal end of the balloon


Ordering Information

TrapLiner® Catheter

The TrapLiner® Catheter is intended for use in conjunction with guide catheters to access discrete regions of the coronary and/or peripheral vasculature, to facilitate placement of interventional devices, and to facilitate the exchange of an interventional device while maintaining the position of a guidewire within the vasculature.

MODEL	SIZE	MINIMUM GUIDE CATHETER SIZE	MAXIMUM GUIDE CATHETER SIZE	PUSH ROD O.D.	GUIDE EXTENSION I.D.	GUIDE EXTENSION O.D.	RAPID EXCHANGE LENGTH	NOMINAL PRESSURE RATING	RATED BURST PRESSURE
5566	6 Fr.	6 Fr. (0.070" / 1.78 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.020" (0.51 mm)	0.056" (1.42 mm)	5.1 Fr. (1.70 mm / 0.067")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa
5567	7 Fr.	7 Fr. (0.078" / 1.98 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.025" (0.64 mm)	0.062" (1.57 mm)	5.7 Fr. (1.90 mm / 0.075")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa
5568	8 Fr.	8 Fr. (0.088" / 2.24 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.025" (0.64 mm)	0.071" (1.80 mm)	6.5 Fr. (2.16 mm / 0.085")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa

Packaged in quantities of 1 unit per box.

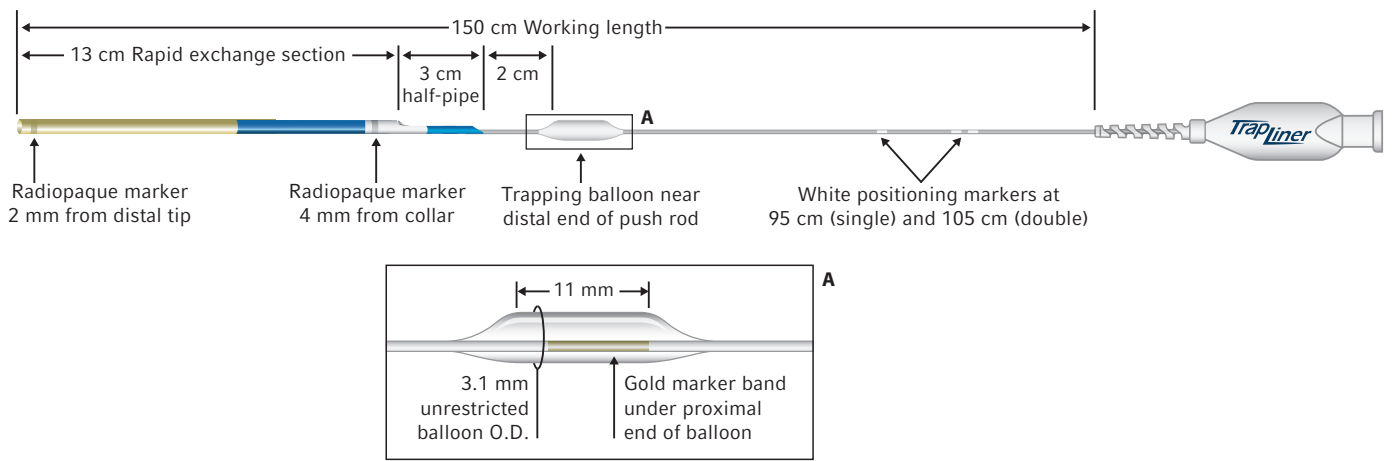

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit teleflex.com.

Teleflex is the home of Arrow®, Deknatel®, Hudson RCI®, LMA®, Pilling®, Rüschi®, and Weck® – trusted brands united by a common sense of purpose.

Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

Regional Offices

United States: Phone +1 919 544 8000, Toll Free 866 246 6990, cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

Latin America: Phone +1 919 433 4999, la.cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

International: Phone +353 (0)9 06 46 08 00, orders.intl@teleflex.com, Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Australia 1300 360 226

Austria +43 (0)1 402 47 72

Belgium +32 (0)2 333 24 60

Canada +1 (0)800 387 9699

China (Shanghai) +86 (0)21 6163 0965

China (Beijing) +86 (0)10 6418 5699

Czech Republic +420 (0)495 759 111

France +33 (0)5 62 18 79 40

Germany +49 (0)7151 406 0

Greece +30 210 67 77 717

India +91 (0)44 2836 5040

Italy +39 0362 58911

Japan +81 (0)3 6632 3600

Korea +82 2 536 7550

Mexico +52 55 5002 3500

Netherlands +31 (0)88 00 215 00

New Zealand 0800 601 100

Poland +48 22 4624032

Portugal +351 22 541 90 85

Singapore (SEA non-direct sales countries) +65 6439 3000

Slovak Republic +421 (0)3377 254 28

South Africa +27 (0)11 807 4887

Spain +34 918 300 451

Switzerland +41 (0)31 818 40 90

United Kingdom +44 (0)1494 53 27 61

For more information, please visit teleflex.com.

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Teleflex, the Teleflex logo, GuideLiner, TrapLiner, Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüschi and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 05/2018.

© 2018 Teleflex Incorporated. All rights reserved. MC-005714 LA EN