


*TrapLiner*<sup>™</sup>  
catheter

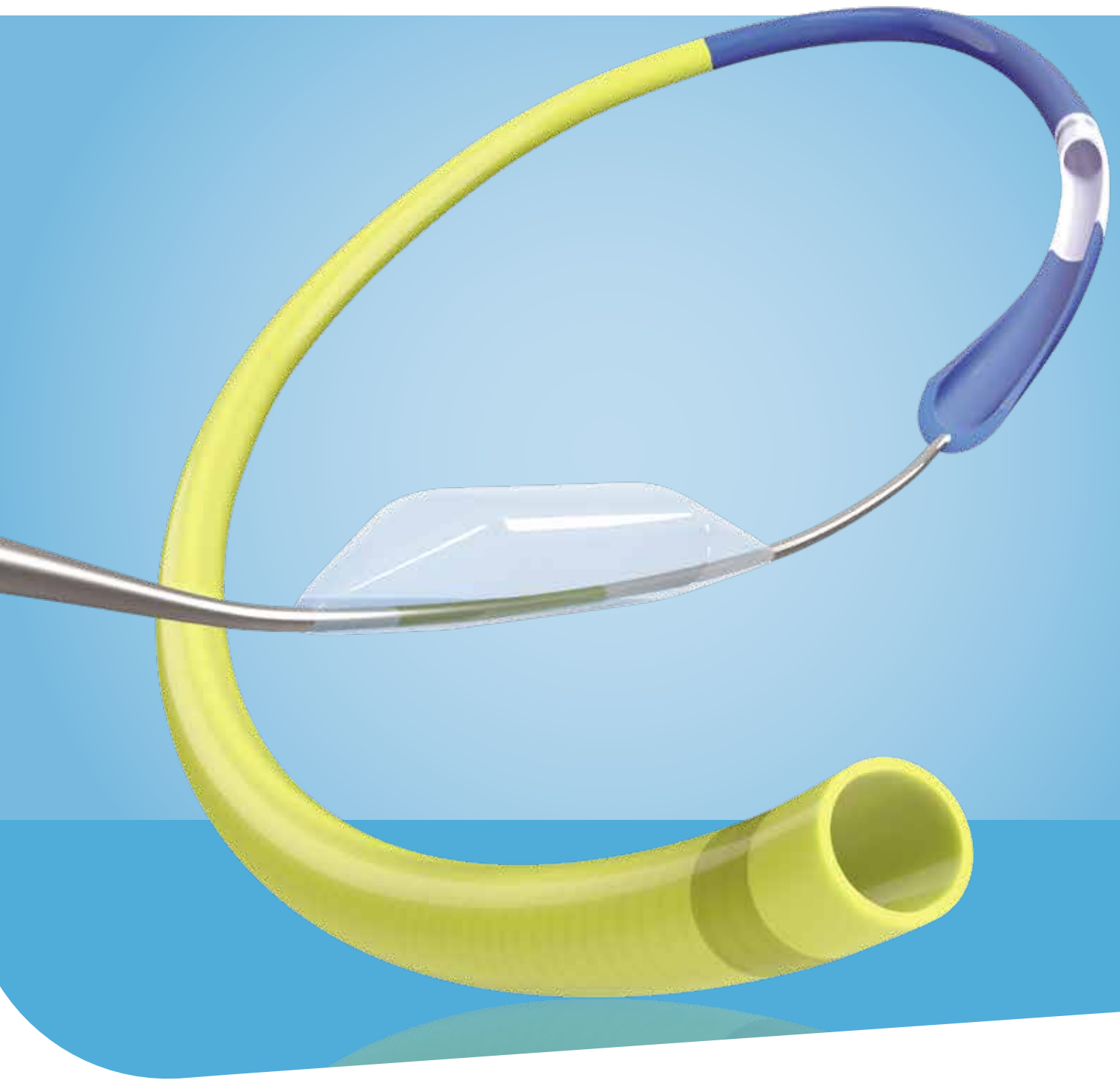

**TrapLiner<sup>®</sup> Catheter**  
Guide Extension Plus Wire Trapping

**Teleflex<sup>®</sup>**

# 2-in-1 Device


## Guide Extension Catheter With Trapping Capabilities


The TrapLiner® Catheter is a rapid exchange guide extension catheter that combines the ability to provide backup support with the ability to trap a 0.014" guidewire against the inner wall of a guide catheter. This functionality helps prevent the loss of guidewire position or unintentional guidewire advancement during catheter exchange, and eliminates the need for alternative catheter exchange techniques.

### Guide Extension Clinical Benefits

Provides coaxial alignment and deep-seating for:

- Added backup support
- Distal device delivery
- Selective contrast delivery

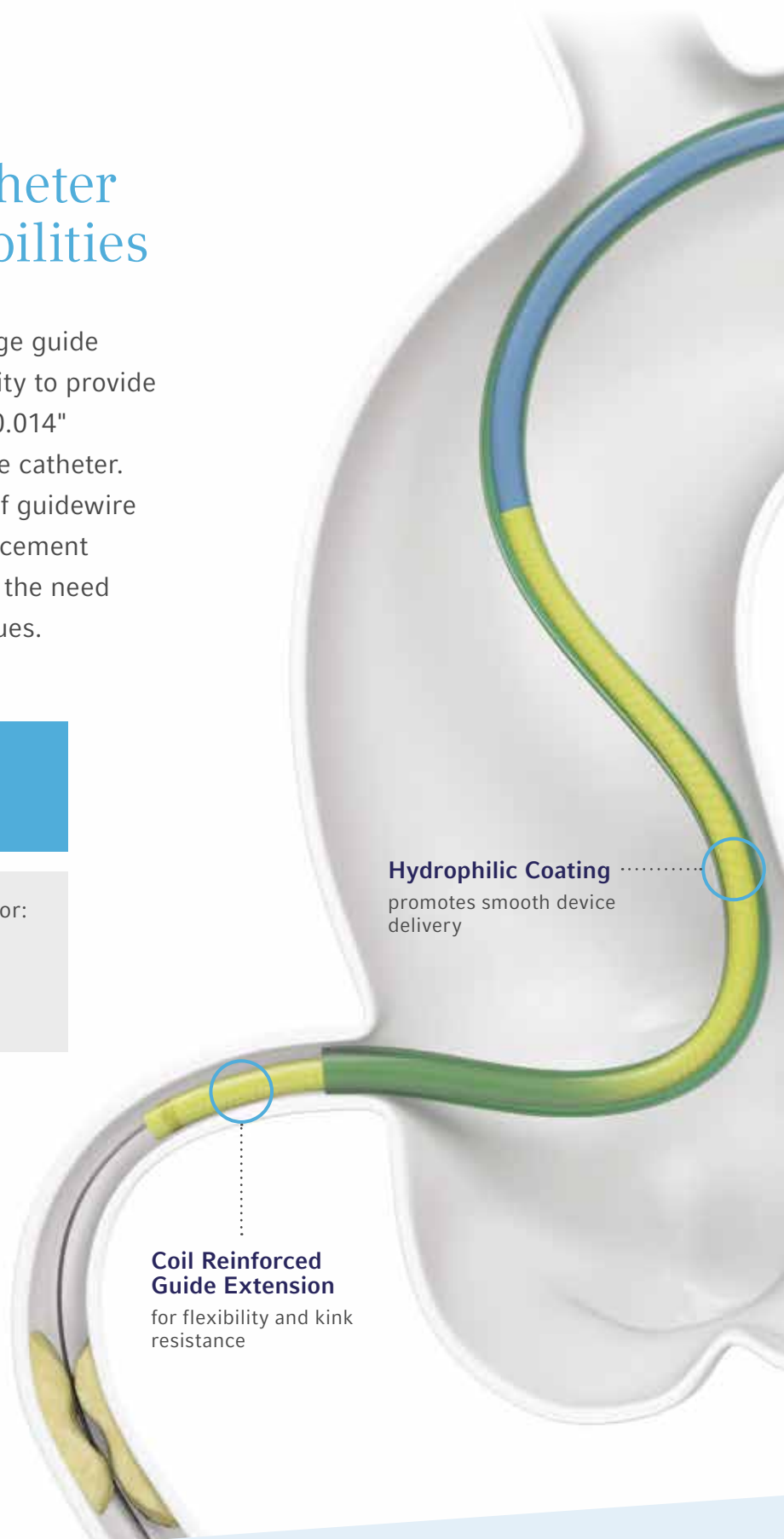

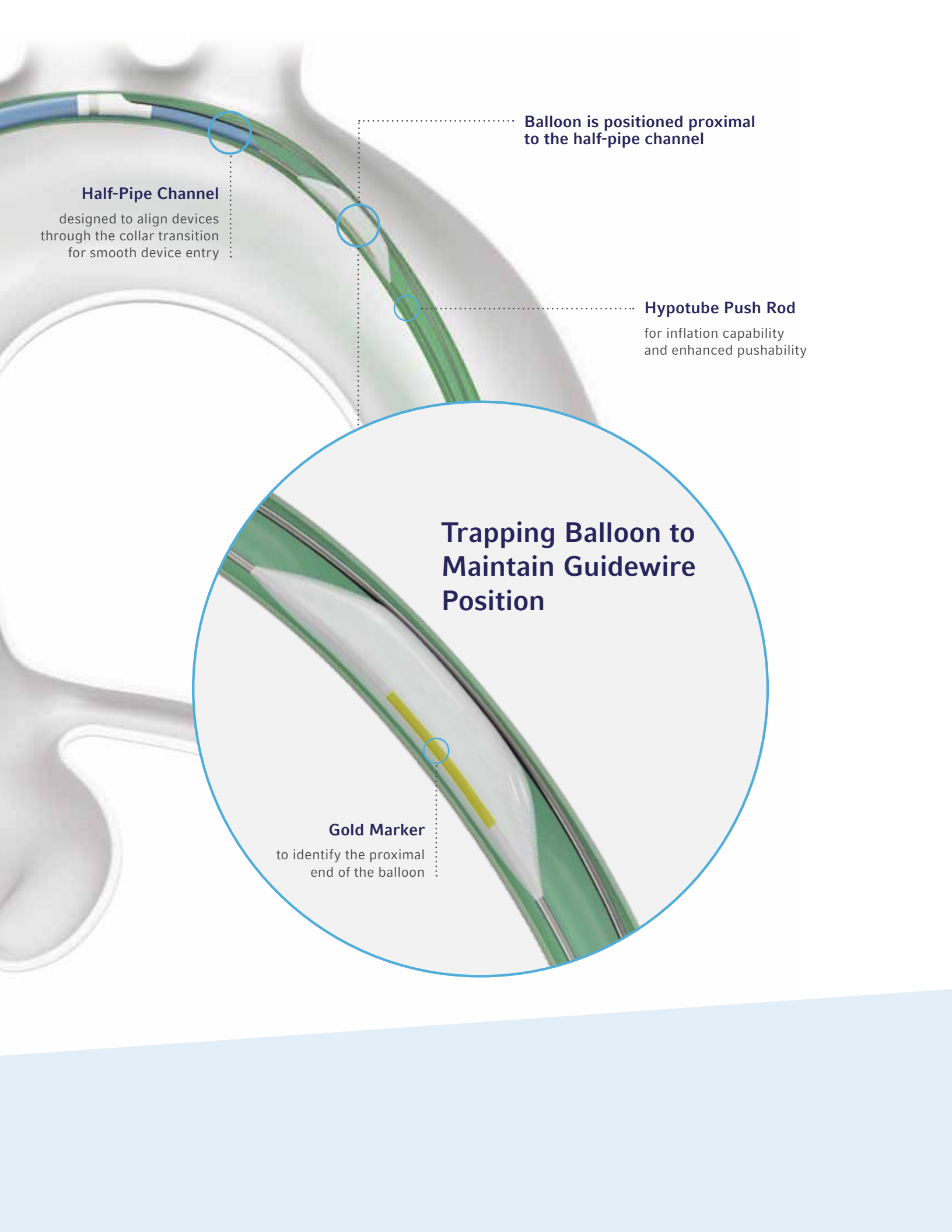

**Half-Pipe Channel**

designed to align devices through the collar transition for smooth device entry

Balloon is positioned proximal to the half-pipe channel

**Hypotube Push Rod**

for inflation capability and enhanced pushability

**Trapping Balloon to Maintain Guidewire Position**

**Gold Marker**

to identify the proximal end of the balloon


# Ordering Information

## TrapLiner® Catheter

The TrapLiner® Catheter is intended for use in conjunction with guide catheters to access discrete regions of the coronary and/or peripheral vasculature, to facilitate placement of interventional devices, and to facilitate the exchange of an interventional device while maintaining the position of a guidewire within the vasculature.

MODEL	SIZE	MINIMUM GUIDE CATHETER SIZE	MAXIMUM GUIDE CATHETER SIZE	PUSH ROD O.D.	GUIDE EXTENSION I.D.	GUIDE EXTENSION O.D.	RAPID EXCHANGE LENGTH	NOMINAL PRESSURE RATING	RATED BURST PRESSURE
5566	6 Fr.	6 Fr. (0.070" / 1.78 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.020" (0.51 mm)	0.056" (1.42 mm)	5.1 Fr. (1.70 mm / 0.067")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa
5567	7 Fr.	7 Fr. (0.078" / 1.98 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.025" (0.64 mm)	0.062" (1.57 mm)	5.7 Fr. (1.90 mm / 0.075")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa
5568	8 Fr.	8 Fr. (0.088" / 2.24 mm I.D.)	8 Fr. (0.091" / 2.31 mm I.D.)	0.025" (0.64 mm)	0.071" (1.80 mm)	6.5 Fr. (2.16 mm / 0.085")	13 cm	12 atm / 1216 kPa	14 atm / 1419 kPa

Packaged in quantities of 1 unit per box.

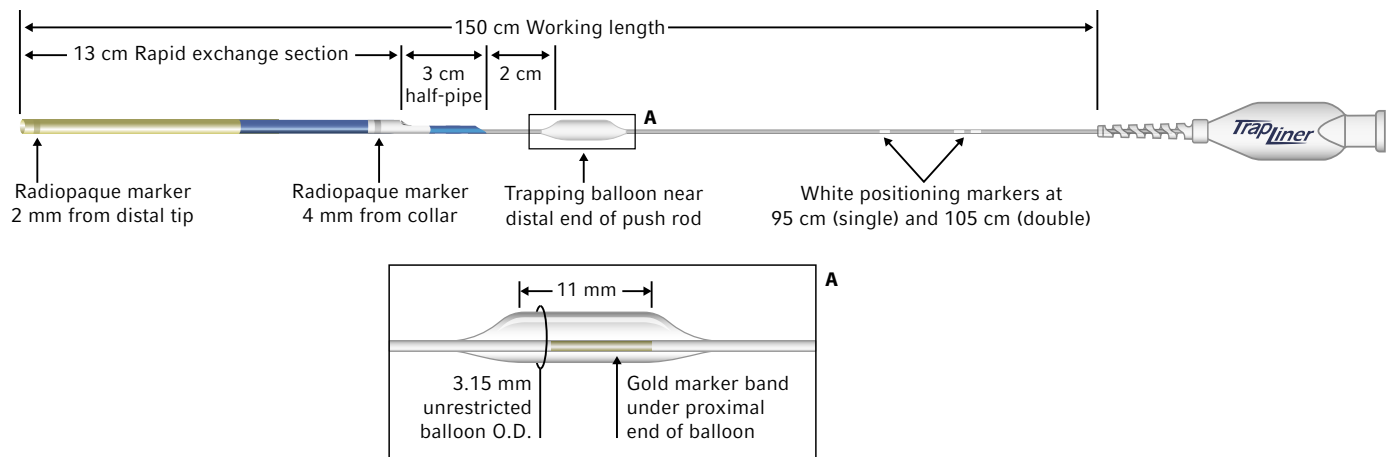


## Feature Comparison Overview

TrapLiner®  
Catheter

○ 6 Fr.   
 ○ 7 Fr.   
 ○ 8 Fr.


Advance guide extension up to 10 cm

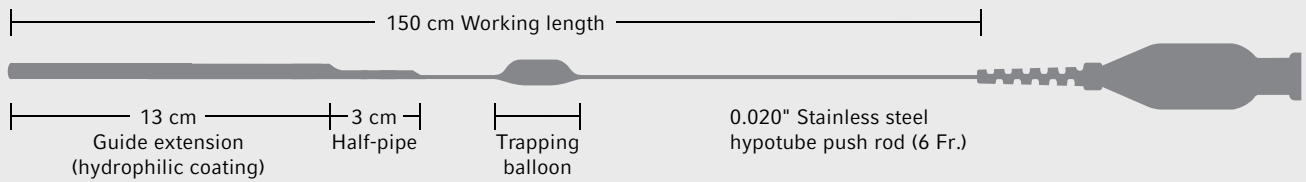

GuideLiner® V3  
Catheter

○ 5 Fr.   
 ○ 5.5 Fr.   
 ○ 6 Fr.   
 ○ 7 Fr.   
 ○ 8 Fr.

Advance guide extension up to 10 cm

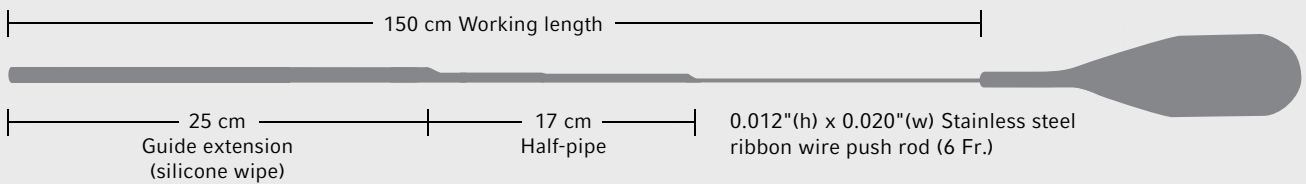

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit [teleflex.com](http://teleflex.com).

Teleflex is the home of Arrow®, Deknatel®, Hudson RCI®, LMA®, Pilling®, Rüsçh®, and Weck® – trusted brands united by a common sense of purpose.

### Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

### Regional Offices

**United States:** Phone +1 919 544 8000, Toll Free 866 246 6990, [cs@teleflex.com](mailto:cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**Latin America:** Phone +1 919 433 4999, [la.cs@teleflex.com](mailto:la.cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**International:** Phone +353 (0)9 06 46 08 00, [orders.intl@teleflex.com](mailto:orders.intl@teleflex.com), Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

**Australia** 1300 360 226

**Austria** +43 (0)1 402 47 72

**Belgium** +32 (0)2 333 24 60

**Canada** +1 (0)800 387 9699

**China** (Shanghai) +86 (0)21 6163 0965

**China** (Beijing) +86 (0)10 6418 5699

**Czech Republic** +420 (0)495 759 111

**France** +33 (0)5 62 18 79 40

**Germany** +49 (0)7151 406 0

**Greece** +30 210 67 77 717

**India** +91 (0)44 2836 5040

**Italy** +39 0362 58911

**Japan** +81 (0)3 6632 3600

**Korea** +82 2 536 7550

**Mexico** +52 55 5002 3500

**Netherlands** +31 (0)88 00 215 00

**New Zealand** 0800 601 100

**Poland** +48 22 4624032

**Portugal** +351 22 541 90 85

**Singapore** (SEA non-direct sales countries) +65 6439 3000

**Slovak Republic** +421 (0)3377 254 28

**South Africa** +27 (0)11 807 4887

**Spain** +34 918 300 451

**Switzerland** +41 (0)31 818 40 90

**United Kingdom** +44 (0)1494 53 27 61

**For more information, please visit [teleflex.com](http://teleflex.com).**

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Teleflex, the Teleflex logo, GuideLiner, TrapLiner, Arrow, Deknatel, Hudson RCI, LMA, Pilling, Rüsçh and Weck are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 05/2018.

© 2018 Teleflex Incorporated. All rights reserved. MC-004101 Rev 0