





# PiggyBack<sup>®</sup> Wire Converter

Convert 0.014" Procedures Into 0.035" Possibilities

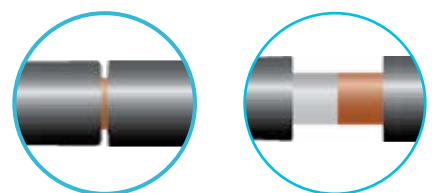

# Transition without Losing Position


The PiggyBack® Wire Converter is a 0.035" outer diameter wire converter that can be delivered over and locked onto a 0.014" guidewire to allow subsequent delivery of 0.035" compatible catheters, sheaths and devices.

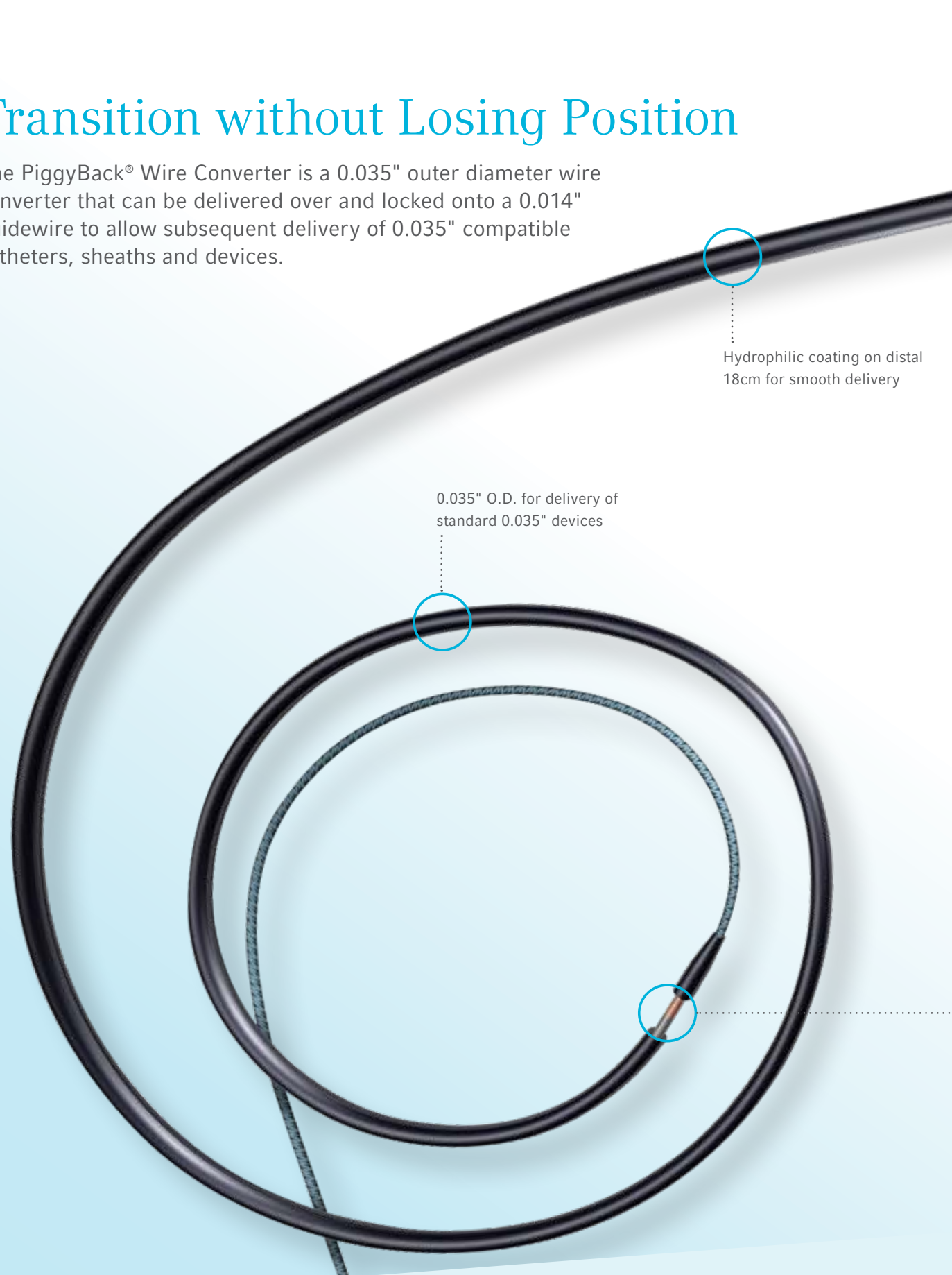

## 0.014" Inside, 0.035" Outside


The PiggyBack® Wire Converter is useful for peripheral procedures requiring a 0.014" guidewire for easier access and a 0.035" guidewire for more support. The PiggyBack® Wire Converter provides the ability to convert a 0.014" guidewire to a 0.035" platform without losing vessel placement or exchanging guidewires.

- For navigating different sized vessels during the same procedure
- For crossing tortuous anatomy

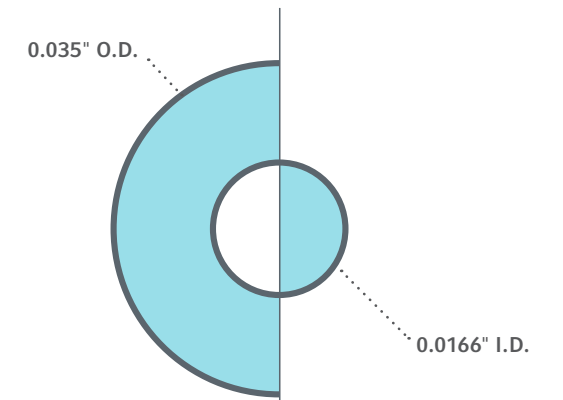

## How Does it Work?

The PiggyBack® Wire Converter can be delivered over a 0.014" guidewire in situ or preloaded onto a 0.014" guidewire before it is placed intravascular.

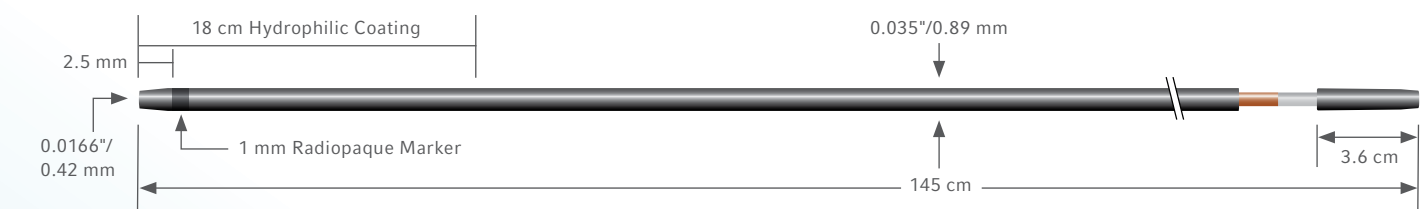

## Specifications and Ordering Information

MODEL	LENGTH	REQUIRED GUIDEWIRE DIAMETER	MINIMUM GUIDEWIRE LENGTH FOR EXCHANGE	MINIMUM INTRODUCER SIZE	DISTAL HYDROPHILIC COATING LENGTH
8802	145 cm	0.014"	290 cm	4F (0.035") I.D.	18 cm

Packaged in quantities of 1 unit per box.

The PiggyBack® Wire Converter is intended to be used with guidewires to access the peripheral vasculature and to facilitate placement of interventional devices.

Teleflex is a global provider of medical technologies designed to improve the health and quality of people's lives. We apply purpose-driven innovation – a relentless pursuit of identifying unmet clinical needs – to benefit patients and healthcare providers. Our portfolio is diverse, with solutions in the fields of vascular and interventional access, surgical, anesthesia, cardiac care, urology, emergency medicine and respiratory care. Teleflex employees worldwide are united in the understanding that what we do every day makes a difference. For more information, please visit [teleflex.com](http://teleflex.com).

Teleflex is the home of Arrow®, Deknatel®, Hudson RCI®, LMA®, Pilling®, Rüschi®, UroLift®, and Weck® – trusted brands united by a common sense of purpose.

### Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

### Regional Offices

**United States:** Phone +1 919 544 8000, Toll Free 866 246 6990, [cs@teleflex.com](mailto:cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**Latin America:** Phone +1 919 433 4999, [la.cs@teleflex.com](mailto:la.cs@teleflex.com), 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

**International:** Phone +353 (0)9 06 46 08 00, [orders.intl@teleflex.com](mailto:orders.intl@teleflex.com), Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

**Australia** 1300 360 226

**Austria** +43 (0)1 402 47 72

**Belgium** +32 (0)2 333 24 60

**Canada** +1 (0)800 387 9699

**China (Shanghai)** +86 (0)21 6163 0965

**China (Beijing)** +86 (0)10 6418 5699

**Czech Republic** +420 (0)495 759 111

**France** +33 (0)5 62 18 79 40

**Germany** +49 (0)711 2090 8000

**Greece** +30 210 67 77 717

**India** +91 (0)44 2836 5040

**Italy** +39 0362 58911

**Japan** +81 (0)3 6632 3600

**Korea** +82 2 536 7550

**Mexico** +52 55 5002 3500

**Netherlands** +31 (0)88 00 215 00

**New Zealand** 0800 601 100

**Poland** +48 22 4624032

**Portugal** +351 22 541 90 85

**Singapore** (SEA non-direct sales countries) +65 6439 3000

**Slovak Republic** +421 (0)3377 254 28

**South Africa** +27 (0)11 807 4887

**Spain** +34 918 300 451

**Switzerland** +41 (0)31 818 40 90

**United Kingdom** +44 (0)1494 53 27 61

**For more information, please visit [teleflex.com](http://teleflex.com).**

Please see the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Teleflex, the Teleflex logo, and PiggyBack are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 03/2020

© 2020 Teleflex Incorporated. All rights reserved. MC-006033 Rev 0