


PROCEDURE GUIDE – GENERAL SURGERY


Traditional Lap Chole

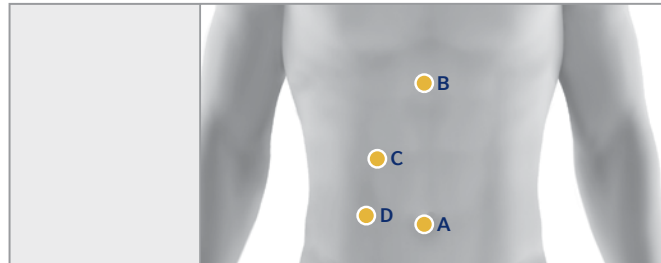

PORT / INSTRUMENT

PORTS: 5 mm or 10 mm ports for scope / instrumentation

MiniLap Chole

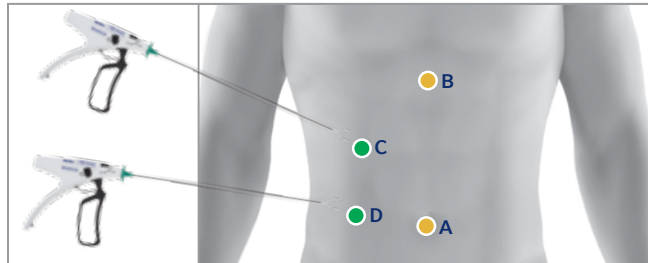


PORT / INSTRUMENT

A / B PORTS: 5mm or 10mm ports for scope / instrumentation

C / D PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

2x3 MiniLap Chole & MiniPolar

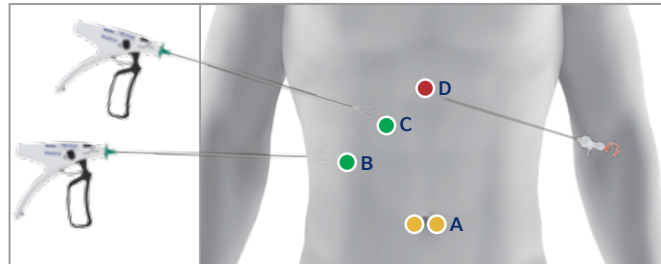

PORT / INSTRUMENT

INSTRUMENTS: Scope, Clip Applier, Bag

A PORT: 5mm or 10mm port for instrumentation

B / C PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

D PORTS: MiniPolar™ Straight Spatula Probe for cautery and dissection

2 x 2 MiniLap Appendectomy




PORT / INSTRUMENT

A / B PORTS: 5mm or 10mm ports for scope / instrumentation

C PORT: MiniGrip® Alligator Grasper for manipulation of appendix


D PORT: MiniGrip® Alligator Grasper, alternate to C Port

PORT COMPARISON

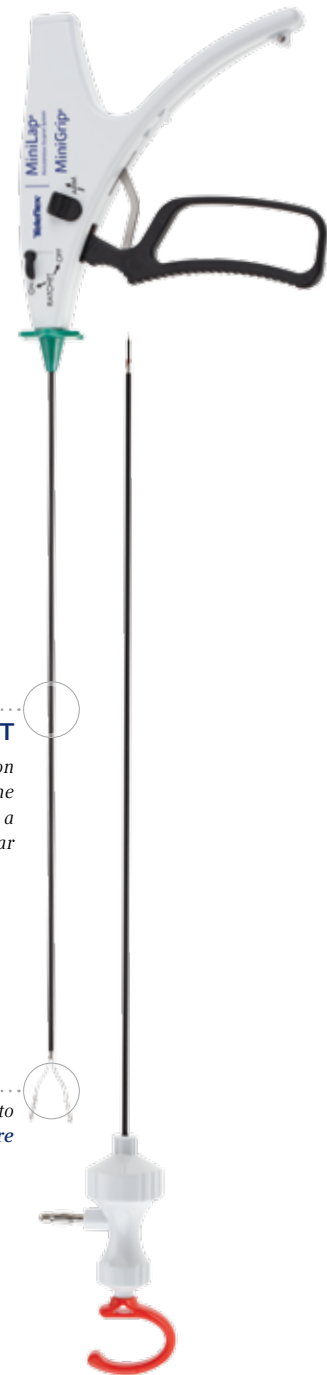

PROCEDURE GUIDE – GYNECOLOGICAL SURGERY

2 Port Advanced Gynecological Procedures Oophorectomy, Hysterectomy, Myomectomy

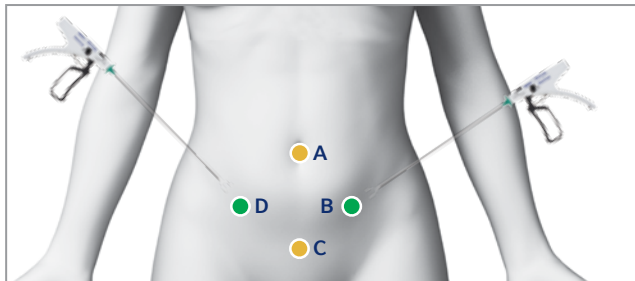


PORT / INSTRUMENT

A PORT: 5mm or 10mm port for scope

B / D PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

C PORT: Energy Device and instrumentation

1x2 MiniLap Excision of Endometriosis/Adhesiolysis

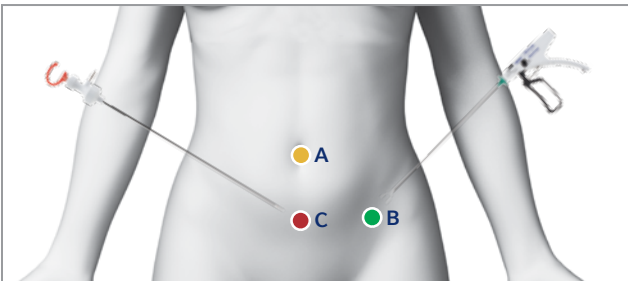


PORT / INSTRUMENT

A PORT: 5mm or 10mm port for scope

B PORTS: MiniGrip® Alligator Grasper for manipulation and retraction

C PORT: Electrosurgery MiniPolar Probe used for ablation of endometriosis

Laparoscopic Ovarian Cyst Removal

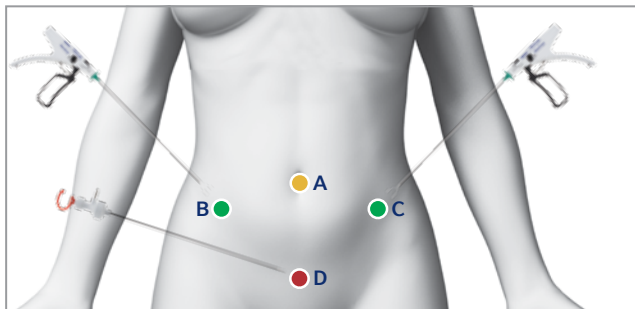

PORT / INSTRUMENT


A PORT: 5mm or 10mm port for scope

Optional second port for instrumentation

B / C PORT: MiniGrip® Alligator Grasper for manipulation and retraction

D PORT: MiniPolar™ Straight Spatula Probe used for cauterization and dissection

Tubal Ligation

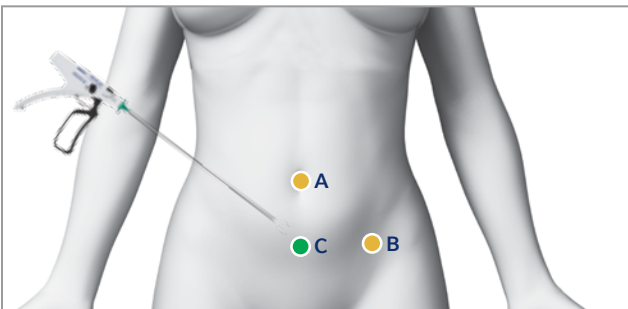

PORT / INSTRUMENT

A / B PORTS: 5mm or 10mm ports for scope/instrumentation

C PORT: MiniGrip® Alligator Grasper used to grasp fallopian tubes


MINILAP® SYSTEM WITH MINIGRIP® HANDLE




Alligator



Babcock

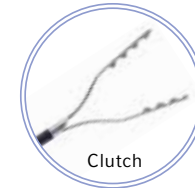

Clutch

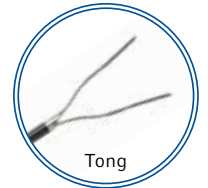

Tong


MINILAP® PERCUTANEOUS GRASPERS




Alligator



Babcock

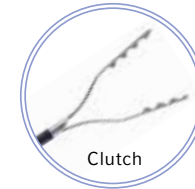

Clutch

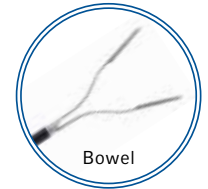

Bowel

MINIPOLAR™ ELECTROSURGERY PROBES



Curved
Spatula Probe



Straight
Spatula Probe



Hook
Probe



Conical
Probe

TO ORDER, CALL TELEFLEX CUSTOMER SERVICE AT 1.866.846.6990

Teleflex, MiniGrip, MiniLap and MiniPolar are trademarks or registered trademarks of Teleflex Incorporated or its affiliates.
© 2015 Teleflex Incorporated. All rights reserved. MC-000827 Rev 1

Teleflex®

TELEFLEX 3015 Carrington Mill Boulevard, Morrisville, NC 27560
Toll Free: 866.246.6990 Phone: +1.919.544.8000
TELEFLEX.COM